



INSTRUCTIONS FOR USE

Read these instructions carefully before you start using evaQwax. Keep this leaflet. You may need to read it again. Do not share evaQwax with any other person. If you require additional information or have any further questions, ask your doctor, pharmacist or nurse. You must see a healthcare professional if your problem worsens or does not improve after 5 days.

What evaQwax is and what it is used for:

evaQwax Ear Spray is a product that is scientifically formulated to help remove earwax from the outer ear canal. evaQwax comes in a 45 ml bottle with a dispensing device. The active ingredient is hydrogen peroxide. This works by releasing oxygen, which breaks the earwax down into smaller fragments that are easier to remove. It may also prevent the need for ear irrigation. In addition to hydrogen peroxide, this product contains a stabiliser that acts as a buffer to maintain a slightly acidic pH, consistent with the physiology of the outer ear canal.

Who can use evaQwax:

evaQwax is suitable for use in adults and children from 3 years old.

There are no known harmful effects when this product is used by pregnant or breastfeeding mothers.

Do not use evaQwax if:

- you are allergic to hydrogen peroxide;
- you suffer from dizziness or any form of otitis (for example: pain, infection, discharge);
- you have, or suspect that you may have, a perforated eardrum/eardrum-related impairment; or
- you suffer from tinnitus and/or have a history of hearing problems.

Using other products or medicine with evaQwax:

evaQwax will not interact with any other product or medicine that you may be taking.

However, do not use it with any other ear products.

How to use evaQwax:


evaQwax is applied using the dispensing pump specially designed for use in the outer ear canal as shown in the diagram below.

For children from the age of 3 to 8 years, the administration has to be especially careful by gently placing the ear adaptor in the ear canal opening.

- Hearing aids must be removed before using evaQwax.
- Spray one to three times to prepare the device.




Tilt your head forwards and gently pull your ear backwards.

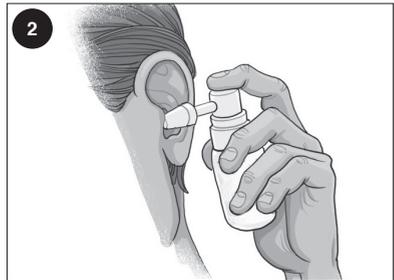

Carefully place the applicator in the outer ear canal and press it 5 times for adults and 2 or 3 times for children aged 3 and up to 8 years. Keep your head tilted forwards for about 5 minutes.



Turn your head in the opposite direction to allow the liquid to drain and dry. Hold a bowl or towel under your ear as the solution drains.

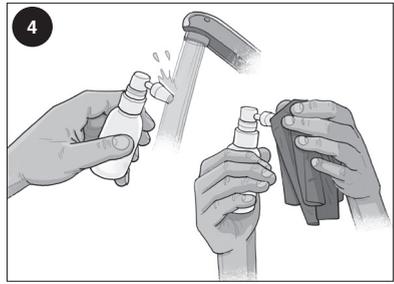

Clean the applicator with water and dry it. Keep it for future applications.

How often to use evaQwax:

Adults and children from the age of 3 years

Wax impaction:

Use twice a day for 3 to 5 days.

Prevention of wax impaction:

If you need to have your ears cleaned out three times per year, use three times per week. If you need to have your ears cleaned out two times per year, use twice a week, and so on.

Possible side effects:

Just because a side effect is stated here, it does not mean that all people using this product will experience that or any other side effect.

It has been reported that ear solutions may cause discomfort, hearing loss, dizziness, skin irritation and tinnitus.

Do not drive or use machines if you experience these effects.

The release of oxygen may cause a temporary bubbling sensation.

Stop using evaQwax if painful or if sensitivities occur.

Precautions:

This product may stain clothing.

In case of contact with eyes, rinse immediately with plenty of water.

Do not swallow.

In case of poisoning, contact the Poison Information Centre: 0861 555 777.

Do not insert foreign objects such as cotton buds or bobby pins into the ear canal.

How to store evaQwax:

Keep out of reach of children.

Store between 2 °C and 30 °C.

Do not refill the container.

Do not freeze.

Do not use after the expiry date stated on the container, even if the container has not been opened.

Return all unused product to your pharmacist.

Do not dispose of unused product in drains or sewerage systems (e.g. toilets).

Composition:

Hydrogen peroxide 3 %; stabiliser.

Date of last revision of the instruction for use:

January 2021

Distributor & Marketer:

Abex Pharmaceutica (Pty) Ltd
Suite C, Rubenstein Ridge
617 Rubenstein Drive
Moreleta Park,
0181
South Africa

Manufacturer:

Mitelos Bioscience S.L.
Parc Tecnològic del Vallès.
Ronda de Can Fatjó 7-B.
08290 Cerdanyola del Vallès.
Barcelona, Spain

ABEX
PHARMACEUTICA (PTY) LTD

Mitelos

CE EVA/IFU/B