



INSTRUCTIONS FOR USE

Read these instructions carefully before you start using evaQwax. Keep this instructions for use, you may need to read it again. Do not share evaQwax with any other person. If you require additional information or have any further questions, ask your doctor, pharmacist or nurse.

What evaQwax is and what it is used for:

evaQwax ear spray is a product that is scientifically formulated to remove impacted earwax from the external ear canal or to help prevent its excessive build-up. evaQwax comes in a 45 ml bottle with an ear dispensing device. The active ingredient is hydrogen peroxide. This works by releasing oxygen, which breaks the earwax down into smaller fragments that are easier to remove. It may also prevent the need for ear irrigation.

In addition to hydrogen peroxide, this product contains a stabiliser that acts as a buffer to maintain a slightly acidic pH, consistent with the physiology of the external ear canal.

Who can use evaQwax:

evaQwax is suitable for use in adults and children from 3 years old.

Do not use evaQwax if:

- you are allergic to hydrogen peroxide;
- you suffer from any form of otitis (for example: pain, infection, discharge);
- you have, or suspect that you may have, a perforated eardrum;
- you have tympanostomy (ventilation) tubes; or
- you have had a recent ear surgery unless specifically cleared by the doctor.

Using other products or medicine with evaQwax:

evaQwax will not interact with any other product or medicine that you may be taking.

However, do not administer it together with any other ear products.

In case of doubt ask your doctor or pharmacist.

How to use evaQwax:


evaQwax is applied using the ear dispensing device specially designed for use in the external ear canal as shown in the drawings below.

If you use hearing aids, you must remove them before using evaQwax.

Before using evaQwax, assemble the ear dispensing device with the bottle by exerting slight pressure, and spray repeatedly until the liquid comes out.




1. Tilt the head so the affected ear faces upward and gently pull the ear backwards.

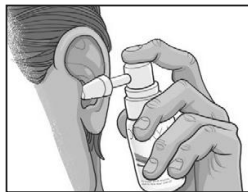

2. Place the ear dispensing device in the external ear canal (with particular care in children below 9 years), and press it as per dosage in section below (**How often to use evaQwax**). Keep the head tilted for about 2 minutes.



3. Turn the head in the opposite direction to allow the liquid to drain and dry holding a clean tissue.

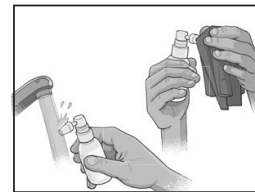

4. Clean the ear dispensing device with water and dry it. Keep it for future applications.

Important: Before each use, spray repeatedly in the air (out of the external ear canal), until the liquid comes out. Then follow the steps indicated in drawings 1 to 4 above.

How often to use evaQwax:

Follow the recommendations in this table, unless your doctor recommends a different schedule.

The schedule of administration varies if evaQwax ear spray is used to treat an already formed earwax blockage, or if you want to prevent its formation. Children from 3 to 9 years old have a smaller external ear canal and they need a smaller quantity of liquid.

		Recommended treatment regimen	
Age	Dosage (No. of sprays)	Earwax blockage	Prevention of excessive earwax build-up
Children from 3 to 9 years old:	Press 2 or 3 times	Twice a day for 3 to 5 days	If you need earwax removal procedure twice per year, use evaQwax twice a week. If you need earwax removal procedure three times per year, use evaQwax three times per week, and so on.
Adults and children over 9 years old:	Press 4 or 5 times	Twice a day for 3 to 5 days	If you need earwax removal procedure twice per year, use evaQwax twice a week. If you need earwax removal procedure three times per year, use evaQwax three times per week, and so on.

Possible side effects:

Just because a side effect is stated here, it does not mean that all people using this product will experience that or any other side effect.

It has been reported that ear solutions may cause discomfort, pain, transient hearing loss (due to a liquid being present in the ear canal), dizziness, skin irritation.

On rare occasions, it has been reported that it can cause itching.

The release of oxygen may cause a temporary bubbling sensation.

In case excessive pain, discomfort or loss of hearing are noticed, as well as any other possible side effects not mentioned before, you should stop using evaQwax and seek medical attention.

Warnings and precautions:

Consult your healthcare professional if the problem worsens or persists.

For external use only, exclusively to be used in the ear.

In case of contact with eyes, rinse immediately with plenty of water.

Do not swallow.

In case of poisoning, contact the Poison Information Centre: 0861 555 777.

This product may stain clothing.

Do not insert foreign objects such as cotton buds or bobby pins into the ear canal.

Pregnant or nursing women, please consult your healthcare professional before use.

Any serious incident that has occurred in relation to the device should be reported to the distributor, and this immediately to the manufacturer. .

How to store evaQwax:

Keep out of reach of children.

Store below 30 °C.

Keep the bottle inside the carton for proper conservation.

Store in an upright position.

Do not refill the container.

Do not freeze.

Do not use after the expiry date printed on the carton and bottle, even if the container has not been opened.

Use within 12 months after opening.

Return all unused product to your pharmacist.

Do not dispose of unused product in drains or sewerage systems (e.g. toilets).

Composition:

Hydrogen peroxide 3 %; stabiliser.

Date of last revision of the instructions for use:

February 2022

Distributed by:

Abex Pharmaceutica (Pty) Ltd
Suite C, Rubenstein Ridge
617 Rubenstein Drive
Moreleta Park, 0181
South Africa
Tel: +27 12 997 6974
Under License from Mitelos Bioscience S.L.



Manufacturer:

Mitelos Bioscience S.L.
Parc Tecnològic del Vallès.
Ronda de Can Fatjó 7-B.
08290 Cerdanyola del Vallès.
Barcelona, Spain

